Vacuum regulator and accessories

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The purpose of the suction regulator is to measure and adjust the vacuum level within the context of medical suction operations. The suction controller has to be connected to the vacuum source, using generally a direct probe on the specific terminal units placed in the hospital or through a rail mounting system.

To use this regulator just press the ON/OFF pushbutton and adjust the vacuum level by the knob as indicated by the pointer on the vacuum gauge till to reach the desired vacuum level. Once the level has been adjusted just press the ON/OFF pushbutton to switch off or to switch on rapidly on the preset vacuum level.

Various accessories are available together with the vacuum regulator to complete the installation and to grant the right use, such as safety and collecting jars, kit for rail installation, probes, terminal tips and silicon tubes.

MAIN FEATURES

- plastic body and cover, preventing risk of corrosion;
- wide membrane continuous suction controller, guaranteeing reliability and fine adjustment;
- available for different vacuum levels: 0-1000, 0-600 and 0-250 mbar/hPa;
- inlet protection valve system in case of inlet positive pressures according to the **UNI EN ISO 10079-3:2000**;
- outlet overpressure (negative) valve for model 250 and 650 mbar;
- vacuum gauge fitted with a protective bumper device;
- on/off pushbutton allowing for the quick restoration of a pre-adjusted vacuum level;
- central adjustment plastic knob;
- available with different probes for for: UNI / AFNOR / DIN / BS standards;
- CE Marked for medical use;

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SAFETY JARS

Safety jar 150 or 500 cc, for the direct connection to the vacuum regulator. Clear MAKROLON body autoclavable at 121°C, equipped with single patient antibacteria filter and overflow safety valve. Serigraphic indication of the maximum fill level. The safety jar does not replace the collecting jar.

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AVAILABLE ACCESSORIES

- Collecting Jars: 1lt 2lt and 4lt - MAKROLON autoclavable at 121°C
- screw caps for 1lt and 2 lt, pressure cap for 4lt
- Disposable sac used with special jars 1lt and 2lt.
- Wall slide to fix the vacuum regulator or the jars on the wall.
- Universal bracket to fix the vacuum regulators or the jars on the rail.
- Silicon hose
- Suction tips
- Catheter holder single or double
- Spare part filters for safety jar

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RAIL FIXATION KIT

The showed kit allows the installation of the vacuum regulator on to the rail. This kit is composed of: 1 aluminium bracket, 1 metallic support with hose connection, 1,5 mt of vacuum hose and 1 probe according to different standards: UNI/AFNOR/DIN /BS.

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SUCTION TROLLEYS

Available in two standard configurations with single or double vacuum regulator. Each trolley is composed of: 5 castors base with vertical support block, single or double vacuum regulator with 150cc safety jars, single or double catheter holder, 3 mt silicon hose with terminal tips, 1,5 mt of vacuum yellow hose. The collecting jars and the specific probe should be ordered separately according to the different needs.

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Vacuum regulator and accessories

Technical data

SUITABLE FOR THE FOLLOWING GAS: Vacuum

INLET DEPRESSION: -100 mbar

ASPIRATION SIDE CONNECTION: G 1/4" M

PATIENT SIDE CONNECTION:

- G 1/2" M

- hose connection Ø 9 mm

DEPRESSION RANGE:

- 0 ÷ -970 mbar (model VR1000DP)

- 0 ÷ -550 mbar (model VR600DP)

- 0 ÷ -190 mbar (model VR250DP)

MAX FLOW-RATE:

- 45 l/min (related to the max inlet depression) for the model VR1000DP

- 45 l/min (related to the max regulation depression with an inlet depression of -900 mbar) for VR600DP e VR250DP

TOLERANCE: 1.6% V.F.S

STORAGE TEMPERATURE: -20 ÷ +60 °C

OPERATING TEMPERATURE: +5 ÷ +35 °C

STANDARD: ISO 7396-1

Product Codes

Vacuum regulator

9478-1212 Vacuum regulator G 1/4 M, 0-1000 mbar with hose connection (model VR1000DP)

9479-1212 Vacuum regulator G 1/4 M, 0-600 mbar with hose connection (model VR600DP)

9480-1212 Vacuum regulator G 1/4 M, 0-250 mbar with hose connection (model VR250DP)

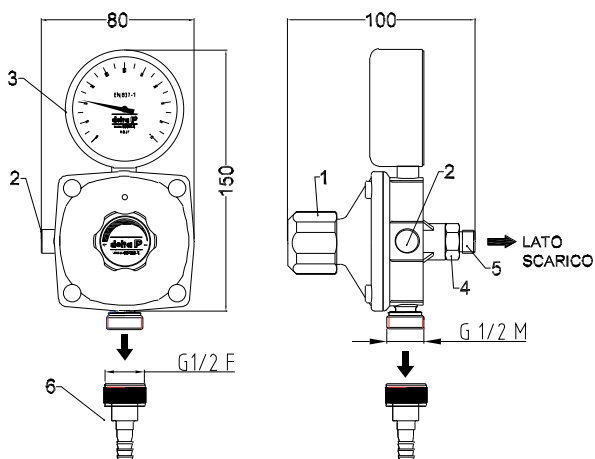
Accessories (to be ordered separately from the vacuum regulator)

9512-1201 150cc safety jar with antibacterial filter


9620-1201 500cc safety jar with antibacterial filter

8903-1010B Antibacterial filters (100 pieces)

Vacuum regulator

Safety jar

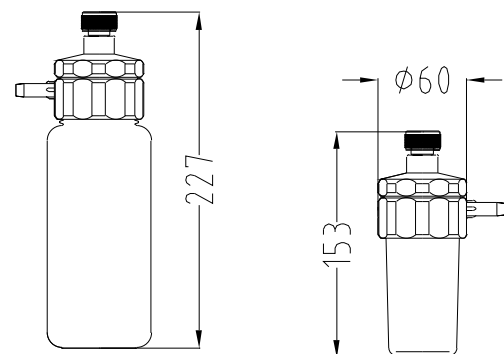LEGEND

1	REGULATION KNOB	4	POSITIVE PRESSURE SAFETY VALVE
2	ON/OFF BUTTON	5	OUTLET CONNECTION
3	VACUUM GAUGE	6	HOSE CONNECTION Ø9

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